

# Medit™ Scanner

Doctors with a Medit scanner can send digital impressions directly to NDX Thoele through the Medit Link.

## Instructions:

- 1) Log in to your Medit Link account at [www.meditlink.com](http://www.meditlink.com). From there, select “PARTNERS”, which is the third item on the left side of the page.
- 2) After accessing the “PARTNERS” page, look for the search bar located in a white box found in the lower middle section of the page. Enter “NDX Thoele” into the search bar and click the magnifying glass or press “Enter” on your keyboard.
- 3) NDX Thoele will appear as the first and only result. Simply click the blue “Request Partnership” button on the far right side.
- 4) A small “Details” window will appear containing contact information for NDX Thoele; click the blue “Request Partnership” button in the lower right corner of the small window.
- 5) A final window will appear confirming your request for partnership; click “Confirm” to complete your connection.

To send us cases, simply select NDX Thoele from the drop-down list of Partners when ordering your cases.



2801 West St. Germain Street. • St. Cloud, MN 56301 • 800.899.1115 • [NDXThoele.com](http://NDXThoele.com)



## Medit™ Scanner

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Thoele, please contact us directly at **800.899.1115** or **[di-thoele@nationaldentex.com](mailto:di-thoele@nationaldentex.com)**