

ORTHO: BLEACHING TRAYS

Before You Begin:

- Before you begin! Help your team member by checking their work before you start yours.
- Read the prescription, verify by signing your agreement on the back of the Rx; you will be held responsible for following the Dr.'s orders.
- Separate and complete rush, remake, and repair cases first.
- Make sure model number matches pan number.
- Verify that every case includes all models, implants or attachment parts, dupes and notes are in pan.
- Make sure work order, design, RX and Dr.'s preferences have been followed precisely.

Quality Standards:

- Read RX and Dr. preferences first.
- Trim trays straight across for regular or to the contour of the teeth for scalloped.
- Smooth edge with a soft felt wheel.
- Ensure that trays are free of holes and bubbles.
- Ensure good fit/no rocking.
- Bleaching trays 1 mm.

Tray Types:

Paint-on

- Allows bleaching of specific areas of selected teeth only

Plain


- Allows all teeth to be bleached



Regular



Regular

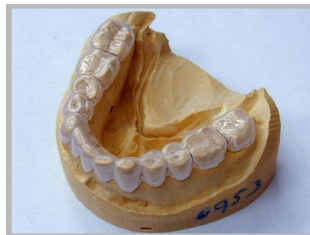

Scalloped



Scalloped



With spacer



With Spacer