

# Comfort H/S Adjustment Information

- For adjustment of the hard outer material it is recommended that you use a pear shaped bur and a high torque-low speed hand piece.
  - For internal adjustments, heat a clean Bard Parker blade and conservatively relieve tight areas.
  - Do NOT place in hot water.
- 

## Comfort H/S Patient Care Information

- Clean with antibacterial soap and warm water.
- Store Dry.
- Avoid any contact with alcohol.
- You can use a Sonic Cleaner or similar type of cleaner once per week
- Do NOT place in hot water.