



Not all patients are candidates for implant therapy. Speak to your dentist to see if dental implants are right for you.



To learn more about nSequence Guided Prosthetics®, please contact 888.809.2777 or visit nsequence.com.



1701 Military Trail, Suite 155A, Jupiter, Florida 33458

Guided Prosthetics and nSequence are registered trademarks of National Dentex, LLC.

MKT00339.RevB © 2022 National Dentex, LLC. All Rights Reserved.



nSequence
Guided Prosthetics®
Improve your Smile.
Improve your Life.



Dental implants can provide you with a better quality of life.

When you're missing many or all of your teeth, you want a dental solution that is customized just for you, returns your smile quickly and reduces your time in the dental office.

With nSequence Guided Prosthetics®:

- » You have fewer appointments than traditional dental implant methods – five appointments as compared to eight*.
- » Surgery time is shortened because your case is planned out in advance
- » You go home after surgery with a same-day smile that is customized just for you.

*Number of appointments may vary.

SMILE TRANSFORMATIONS



Before



After



Before



After



Before



After



Before



After

Photos courtesy of Tarek Assi, DMD, FAAID, DICOI, DABOI/ID from Coral Springs, FL

HERE'S HOW NSEQUENCE® WORKS!

It is possible to regain a natural, beautiful smile in as few as 4-5 appointments.

Appointment #1

Your dentist will gather scans, photos, tooth shades and an impression of your smile. These records are sent digitally to NDX nSequence, where a custom set of teeth are scientifically designed for your mouth and facial features so that they have a personalized fit.

Appointment #2

Your surgeon places the implants in just a few hours so you can leave the office with a beautiful temporary implant-supported restoration that looks, feels and functions like real teeth. This allows you to chew, smile and live your life normally.

Appointment #3

Your dentist checks your mouth to ensure that you are ready for your final implant-supported teeth and sends all of your final information to the laboratory.

Appointment #4

You and your dentists check a prototype of your implant-supported restoration to ensure that it looks good and functions well.

Appointment #5

Your final implant-supported restoration is placed.

It's that simple!