



With our PMMA Temporaries, you can provide your patients with a beautiful smile as soon as you prepare their teeth. These crowns and full-arch bridges restore lost vertical dimension and promote contoured healing for ovate pontics or anterior extractions. The temporary material is designed using CAD/CAM and forms a tooth shape that protects the prepared tooth, preventing damage to the periodontal tissues (gums) and disguises the prepared tooth to a degree until the final restoration can be made.

Benefits

- 100% biocompatibility
- High resistance to abrasion
- Reduce chairside preparation
- Prevents damage to the periodontal tissues

Preparation

- 1.5mm occlusal and axial wall reduction
- 1mm chamfer margin required

Adjusting and Polishing

- Use a fine grit diamond at slow speed with copious amounts of water
- Polish with a porcelain polishing wheel

What to Send

- Upper and lower models, impressions, or digital scans
- Bite Registration
- Shade

Insurance Codes

- D2799 Provisional crown– further treatment or completion of diagnosis necessary prior to final impression

Please allow 5-7 days turnaround time