

PATIENT INFORMATION

Tips on caring for your Crystal Clear®

QUICK CLEANING

- Your Crystal Clear is non-porous and will not absorb stains or odors.
- You may use a Sonic Cleaner and Retainer Brite tablets daily OR
- Simply brush it with your toothbrush and warm water each time you remove it.
- Do not use alcohol or alcohol-based products to clean your splint.
- Never boil your Crystal Clear.

EASY STORAGE

- Use your blue Crystal Clear case to store your new splint, wet or dry.
- Do not use alcohol or alcohol-based products. Many mouthwashes contain alcohol so please read the label carefully!

Please call your dentist with any questions
regarding your Crystal Clear.