

Benefits

- Durable: Made with clinically unbreakable Crystal Clear 450®
- Design: Anterior opening for patient comfort, better air flow and additional tongue space.
- Retention: No ball clasps required - the combination of the design and Crystal Clear 450 material achieves sufficient retention.

Indications

ClearDream™ is intended to reduce night time snoring and mild to moderate Obstructive Sleep Apnea (OSA) in adults.

Contraindications

This device is contraindicated for patients who:

- have central sleep apnea
- have severe respiratory disorders
- have loose teeth or advanced periodontal disease
- are under 18 years of age
- missing 1st and 2nd molars in any quadrant

Material Properties

ClearDream is made using proprietary, clinically unbreakable Crystal Clear 450 material with a design/wax/inject manufacturing process that allows the lower arch with fins to be fabricated in one piece, greatly reducing the likelihood of fin breakage.

What to Send

- Full upper and lower VPS impressions or digital scans.
- A Bite registration - A George Gauge Bite is recommended. If you do not have one available, the ClearDream Bite Technique is acceptable, please see the bite technique instruction sheet. An accurate bite is essential for fabrication.

Medical Insurance Codes

- E0486 Oral Appliance to Treat Sleep Apnea
- G47.33 Diagnostic Code: Obstructive Sleep Apnea

If you are going to file for insurance to cover the costs of the ClearDream, file for medical insurance, not dental insurance. These codes have all had successful prior approval. Be sure to have a note of medical necessity signed by the prescriber and a description of the ClearDream and its function.