

## Indications

- Prevent damage from parafunction
- Protect restorations
- Diagnose occlusal concerns

## Contraindications

- Advanced periodontal disease
- Severe bruxism
- Multiple edentulous areas
- Short clinical crown, minimal retentive undercut

## Benefits of Comfort H/S

- Durable: Protects natural teeth and restorations from the damaging effects of parafunction
- Comfortable: Soft interior provides maximum comfort to increase patient compliance
- Cost Effective: Increase case acceptance with a low lab fee

## Adjustment Tips

- For adjustment of the hard outer material, it is recommended that you use a pear shaped bur and a high torque - low speed handpiece
- For internal adjustments, heat a surgical blade and conservatively relieve tight areas

## Patient Care

- Clean with antibacterial soap and cool water
- Store dry
- Do not use alcohol containing cleaners or breath fresheners

## Taking the Bite Registration

- Place two cotton rolls behind the cuspids and instruct the patient to close until resistance is felt
- With the patient closed into this open bite, inject bite registration material into the posterior openings of both quadrants
- Inject bite registration material into the anterior opening

## What to Send

- Upper and lower VPS impressions
- Open bite with 2mm clearance in the posterior