


NTI-tss PlusTM Soft

A more comfortable NTI-tss Plus


NDX[®] NATIONAL DENTEX LABS

For more information,
please call 800.678.4140

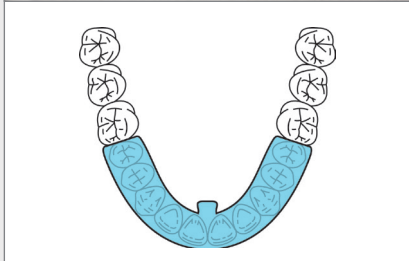

Simple Steps to a Maximum Protrusive Measurement

- Instruct the patient to protrude their mandible as far as possible
- While in the most protrusive position, use a ruler to measure from the labio-incisal of upper centrals to lingual-incisal of lower incisors

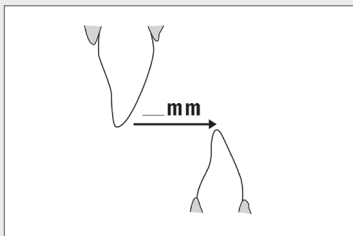

Features & Benefits

- Save Chair Time: small size and soft liner makes it easier to seat
- High patient compliance, increased comfort
- Less pressure on anterior restorations or compromised natural dentition

Indications

- Extreme crowding
- Veneers
- Worn restorations or natural dentition
- Sensitive teeth

Contraindications

- Advanced periodontal disease
- Severely worn centrals
- Incompatible occlusal scheme

Patient Care Instructions

- Simply run the device under warm water (no toothpaste) each time you remove it
- Do not use alcohol or alcohol-based products for soaking or cleaning
- Never boil the NTI-tss Plus Soft

What to Send

- Full arch VPS impressions, models or digital files
- Maximum protrusive measurement

Insurance Codes

All plans are different so the coverage may vary. In general, the same insurance codes for splints, TMJ appliances, habit appliances all apply to the NTI-tss Plus.

Some useful codes are:

- D9940: Occlusal Guard, by report
- D7880: Occlusal Orthotic Device, by report
- D8210: Removable Habit Appliance

For Medical Submission:

- CPT 21110-52 Interdental Fixation Device/D 7880 Orthotic Appliance
- 2015/16 ICD-10-CM M26.60 Temporomandibular joint disorder, unspecified
- 2015/16 ICD-10-CM M26.69 Other specified disorders of temporomandibular joint

Turnaround Times

Please allow 2 weeks for case turnaround.

NTITMTSS
YOUR MIGRAINE GUARD