



Features & Benefit

- Alleviate Sleep Apnea: Research shows it is the most successful oral appliance Obstructive Sleep Apnea treatment available
- Reduce snoring: More than 96% of patients using the appliance have stopped snoring
- Improve patient's sleep: The quality of their sleep has a huge impact on the overall quality of your patient's life
- Increase lateral and protrusive range of motion for increased patient comfort and compliance

Indications


- Anti-snoring
- Indicated for Obstructive Sleep Apnea
- CPAP Intolerance

Contraindications

- Loose teeth or dentures
- Central Sleep Apnea

What to Send

- Upper and lower VPS impressions or digital file
- Centric bite registration
- Maximum protrusive measurement

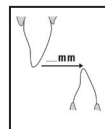

Simple Steps to a Maximum Protrusive Measurement

- Instruct the patient to protrude their mandible as far as possible
- While in the most protrusive position, use a ruler to measure from the labio-incisal of upper centrals to lingual-incisal of lower incisors



TAP[®] 3

Medical Insurance Codes

- G47.33 Diagnostic Code: Obstructive Sleep Apnea
- E0486: Oral Appliance to Treat Sleep Apnea (Medicare Approved)

If you are going to file for insurance to cover the costs of the TAP[®] 3, file for medical insurance, not dental insurance.

These codes have all had successful prior approval. Be sure to have a note of medical necessity signed by the prescriber and a description of the TAP[®] 3 and its function.