

Verotek™ Layered

Technical bulletin from National Dentex
For more information, please call 800.678.4140

Indications

- Anterior and Posterior Crowns
- Bridges (unlimited span)
- Suitable for implants

Contraindications

- Feather margin is acceptable but a chamfer is preferred
- Tighter inter-occlusal space where core may be exposed

Benefits

- Made with biocompatible medical-grade zirconia
- Fabricated using CAD/CAM technology for consistent fit
- Excellent shade match to natural dentition

Preparation

- Chamfer margin preferred
- 1.5-mm occlusal reduction is preferred
- 1.5-mm axial wall reduction is ideal

Cementing Recommendations

- Cement with a Resin or Resin Modified Glass Ionomer

What to Send

- Upper and lower models, impressions or digital scans
- Shade
- Bite Registration

Insurance Codes

- D2740- Crown - Porcelain/Ceramic Substrate
- D6245- Pontic Porcelain/Ceramic
- D6740- Abutment Crown Porcelain/Ceramic

All Verotek restorations have a lifetime guarantee.

NDX[®]  **NATIONAL
DENTEX LABS**
The Network of Artisans and Innovators