

Important ThermoFit Information for the Doctor

Hot Tip for Inserting ThermoFit:

Immediately prior to insertion, immerse the ThermoFit in hot tap water for one to two minutes. This simple step will allow final customization for the patient's optimum comfort.

Benefits of ThermoFit:

- The splint is self-adjusting. Heat it under hot water and the guard will mold to the teeth
- Less Bulky than a traditional hard/soft guard
- Better fit reduces chair time
- Superior wearability and patient comfort
- Amine-Free -- It won't yellow with age, always transparent
- Hypoallergenic - monomer and Bisphenol A free

ThermoFit Patient Care

- 1) Before insertion: Run under hot water to regain flexibility.
- 2) Your ThermoFit may be stored dry. When stored in water it will become cloudy but will become clear again when it dries.
- 3) Clean daily with a "medium soft" tooth brush and water. A non-alcohol based cleanser may be used once a week, not to exceed five minutes.
- 4) You can also use a Sonic Cleaner with Retainer Brite tablets weekly.