


# Quick Reference Guide: How to Export a File for NDX from MyiTero.com


Steps for Crown and Bridge or Implant Cases

- 1 Log into **www.myitero.com** on your computer.  
\*Same user name and password used to log into the scanner.

- 2 Go to **Patients** tab.

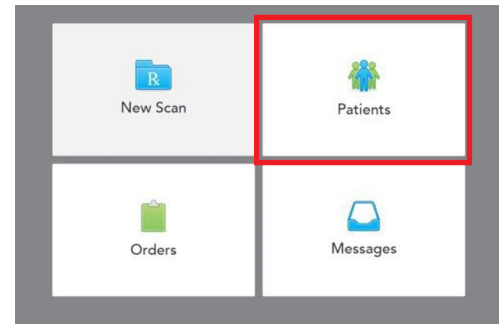


- 3 Locate your patient's information page. Click **Export**.

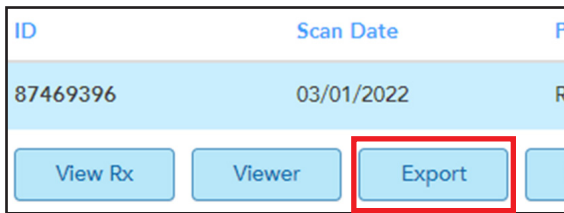


- 4 Expand the CAD/CAM System option.




- 5 Toward the bottom of the list, select: **Color Export - Open Model**

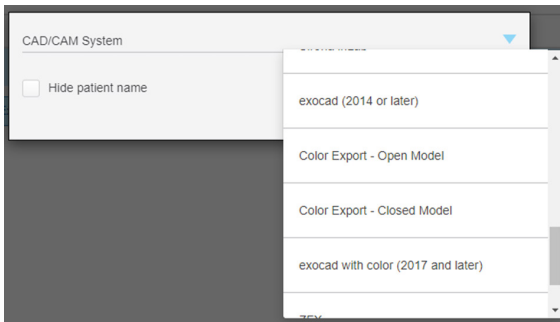

- 6 Choose **Export**.

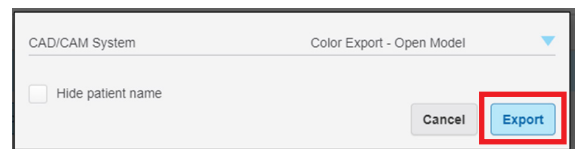

- 7 A zip file of the case will be saved to the download folder used by your browser.

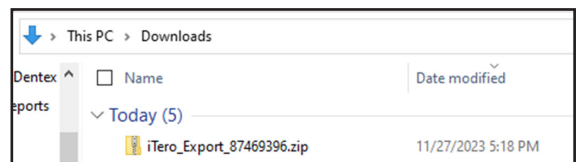


# Quick Reference Guide: How to Export a File for NDX from MyiTero.com


Steps for iRecord, Ortho and Denture Cases

**1** Log into **www.myitero.com** on your computer.  
\*Same user name and password used to log into the scanner.

**2** Go to **Patients** tab.

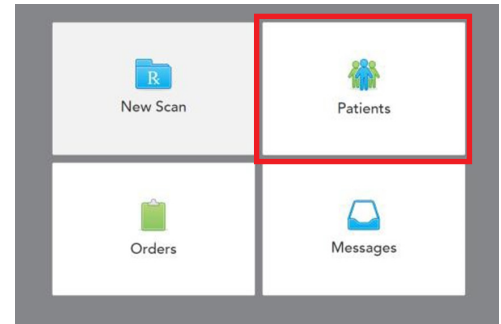

**3** Locate your patient's information page. Click **Export**.

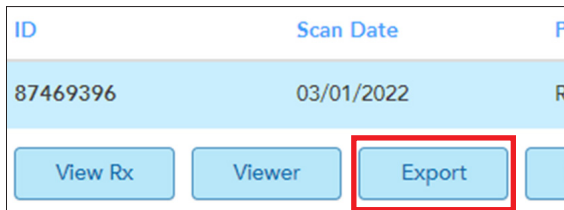

**4** Change the export preferences to those shown in the image below. It is important to match this configuration for best results at the lab:

- Open Shell
- File per Arch
- PLY (Color)



**5** Choose **Export**.



**6** A zip file of the case will be saved to the download folder used by your browser.

