

Medit[®] Scanner

Offices with a Medit scanner can send digital impressions directly to NDX through the Medit Link.

Instructions:

- ① Log in to your *Medit Link* account at www.meditlink.com. From there, select *Partners*, which is the third item on the left side of the page.
- ② After accessing the *Partners* page, look for the search bar located in a white box found in the lower middle section of the page. Enter **your preferred NDX lab** into the search bar and click the magnifying glass or press *Enter* on your keyboard.
- ③ **Your preferred NDX lab** will appear as the first and only result. Simply click the blue *Request Partnership* button icon on the far right side.
- ④ A small *Details* window will appear containing contact information for **your preferred NDX lab**; click the blue *Request Partnership* button in the lower right corner of the small window.
- ⑤ A final window will appear confirming your request for partnership; click *Confirm* to complete your connection.

To send us cases, simply select **your preferred NDX lab** from the drop-down list of *Partners* when ordering your cases.



For questions, please contact Kelly Bevington, RDA, EFDA at kelly.bevington@nationaldentex.com or call her at 724.244.9499.