

# Medit<sup>®</sup> Scanner

Offices with a Medit scanner can send digital impressions directly to NDX nSequence through Medit Link.

## Instructions:

- 1 Log in to your *Medit Link* account at [www.meditlink.com](http://www.meditlink.com). From there, select *Partners*, which is the third item on the left side of the page.
- 2 After accessing the *Partners* page, look for the search bar located in a white box found in the lower middle section of the page. Enter **NDX nSequence** into the search bar and click the magnifying glass or press *Enter* on your keyboard.
- 3 **NDX nSequence** will appear as the first and only result. Simply click the blue *Request Partnership* button icon on the far right side.
- 4 A small *Details* window will appear containing contact information for **NDX nSequence**; click the blue *Request Partnership* button in the lower right corner of the small window.
- 5 A final window will appear confirming your request for partnership; click *Confirm* to complete your connection.

To send us cases, simply select **NDX nSequence** from the drop-down list of *Partners* when ordering your cases.



For questions, please contact your NDX nSequence Sales Specialist.