

3Shape Trios®

Offices with a TRIOS scanner can send digital impressions directly to NDX nSequence through 3Shape Communicate.

To send a file:

Dentists should be registered with a 3Shape Communicate account when the scanner is installed. If NDX nSequence is not already pre-installed on the list of available labs, send a *Connection Request* from the *Communicate* account to NDX.

Note: Dentists can register themselves at <http://portal.3ShapeCommunicate>.

NDX nSequence will respond that the lab is TRIOS ready to receive your scans.

- 1 After selecting the patient name, press *New Session*.
- 2 The lab list will appear, select **NDX nSequence**.
- 3 Complete the order form information. In the *Comments* section, specify specific materials and if the case is modelless.
- 4 Complete the scanning process and press the *Send Order Envelope Icon*.
- 5 On the *Send Order* screen, verify the case information and scans, and then press *Send Order*.



For questions, please contact your NDX nSequence Sales Specialist.