

To send a file:

iTero Element:

- 1 Add **NDX nSequence (ID # 1574)** by calling *iTero Support* at 800.577.8767.
- 2 Start a new Rx, and after entering patient name and case type, select **NDX nSequence** from the *Ship To* menu.
- 3 If modelless, specify in the notes.
- 4 After scanning, press the *Envelope Icon*, add signature,

iTero HD2.9 and Previous Models

- 1 Add **NDX nSequence (ID # 1574)** by calling *iTero Support* at 800.577.8767 or from within your iTero online account.
- 2 Start a new Rx, enter required information and select **NDX nSequence** from the *Ship To* menu.
- 3 If modelless, specify in the notes.
- 4 After completing the scan process, press the *Send* button.



For questions, please contact your NDX nSequence Sales Specialist.