



Doctors with an iTero® scanner can send digital impressions directly to NDX Affinity through iTero online.

To send a file:

iTero® Element™

- 1) Add **NDX Affinity** (ID # **10575**) by calling iTero Support 800-577-8767.
- 2) Start a new Rx, and after entering patient name and case type, select *NDX Affinity* from the Ship To menu.
- 3) If model-less, specify in the notes.
- 4) After scanning, press the *Envelope icon*, add signature, and press *Confirm & Send*.

iTero® HD2.9 and Previous Models

- 1) Add **NDX Affinity** (ID# **10575**) by calling iTero Support at 800-577-8767 or from within your iTero online account.
- 2) Start a new Rx, enter required information and select *NDX Affinity* from the Ship To menu.
- 3) If model-less, specify in the notes.
- 4) After completing the scan process, press the *Send* button.



NDX[®]  Affinity
YOUR DENTAL LAB PARTNER

iTero[®]

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Affinity, please contact us directly at **800.292.1894** or **di-Affinity@NationalDentex.com**.