

Midmark Mobile True Definition™ Scanner

Doctors with a Midmark Mobile True Definition™ scanner can send digital impressions directly to NDX Albensi through the Midmark Connection Center.

To send a file:

- 1) Log in to your Midmark Connection Center Account that was created when your scanner was installed.
If assistance is needed, please contact Midmark Digital Impression Customer Service at 937.526.8387.
- 2) Verify that NDX Albensi and NDX Albensi (No Model) are available under the Scanner Laboratory menu.
 - If not, please contact Midmark Digital Impression Customer Service (937.526.8387) and request that these profiles be added to the scanner menu.
- 3) Select the NDX Albensi profile from the drop-down menu that matches your case preference:
 - NDX Albensi (choose this option when requesting models)
 - NDX Albensi (No Model*) (choose this option when requesting modeless restorations)
- 4) Complete the intraoral scan and select the NDX Albensi profile that meets your model preference.
- 5) Complete the True Definition online Rx.
- 6) Upload the scan data to the Midmark Communication Center.

* Dentists may request a modeless monolithic restoration by selecting the NDX Albensi (No Model) profile that has been previously added to the scanner menu (see step 2 above).



100 Colony Drive • Irwin, PA 15642 • 800.734.3064 • NDXAlbensiLab.com



NDX[®]  Albensi

Midmark Mobile True Definition[™] Scanner

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Albensi, please contact us directly at **800.734.3064** or iosfiles@albensilab.com