



Doctors with an iTero® scanner can send digital impressions directly to NDX D.H. Baker through iTero online.

To send a file:

iTero® Element™

- 1) Add **NDX D.H. Baker** (ID # **2033**) by calling iTero Support 800-577-8767.
- 2) Start a new Rx, and after entering patient name and case type, select NDX D.H. Baker from the Ship To menu.
- 3) If model-less, specify in the notes.
- 4) After scanning, press the Envelope icon, add signature, and press Confirm & Send.

iTero® HD2.9 and Previous Models

- 1) Add **NDX D.H. Baker** (ID# **2033**) by calling iTero Support at 800-577-8767 or from within your iTero online account.
- 2) Start a new Rx, enter required information and select NDX D.H. Baker from the Ship To menu.
- 3) If model-less, specify in the notes.
- 4) After completing the scan process, press the Send button.



iTero®

NDX® D.H.Baker
HIGH AESTHETIC DENTAL RESTORATIONS

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX D.H. Baker, please contact us directly at **800.946.8880** or **di-DHBaker@NationalDentex.com**.