

# Midmark Mobile True Definition™ Scanner

Doctors with a Midmark Mobile True Definition™ scanner can send digital impressions directly to NDX Dodd through the Midmark Connection Center.

## To send a file:

- 1) Log in to your Midmark Connection Center Account that was created when your scanner was installed.  
If assistance is needed, please contact Midmark Digital Impression Customer Service at 937.526.8387.
- 2) Verify that NDX Dodd and NDX Dodd (No Model) are available under the Scanner Laboratory menu.
  - If not, please contact Midmark Digital Impression Customer Service (937.526.8387) and request that these profiles be added to the scanner menu.
- 3) Select the NDX Dodd profile from the drop-down menu that matches your case preference:
  - NDX Dodd (choose this option when requesting models)
  - NDX Dodd (No Model\*) (choose this option when requesting modeless restorations)
- 4) Complete the intraoral scan and select the NDX Dodd profile that meets your model preference.
- 5) Complete the True Definition online Rx.
- 6) Upload the scan data to the Midmark Communication Center.

\* Dentists may request a modeless monolithic restoration by selecting the NDX Dodd (No Model) profile that has been previously added to the scanner menu (see step 2 above).



24 Lukens Drive • New Castle, DE 19720 • 800.441.9005 • [NDXDodd.com](http://NDXDodd.com)



NDX<sup>®</sup>  Dodd

# Midmark Mobile True Definition<sup>™</sup> Scanner

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Dodd, please contact us directly at **800.441.9005** or **[di-dodd@nationaldentex.com](mailto:di-dodd@nationaldentex.com)**