

# Midmark Mobile True Definition™ Scanner

Doctors with a Midmark Mobile True Definition™ scanner can send digital impressions directly to NDX Florida through the Midmark Connection Center.

## To send a file:

- 1) Log in to your Midmark Connection Center Account that was created when your scanner was installed.  
If assistance is needed, please contact Midmark Digital Impression Customer Service at 937.526.8387.
- 2) Verify that NDX Florida and NDX Florida (No Model) are available under the Scanner Laboratory menu.
  - If not, please contact Midmark Digital Impression Customer Service (937.526.8387) and request that these profiles be added to the scanner menu.
- 3) Select the NDX Florida profile from the drop-down menu that matches your case preference:
  - NDX Florida (choose this option when requesting models)
  - NDX Florida (No Model\*) (choose this option when requesting modeless restorations)
- 4) Complete the intraoral scan and select the NDX Florida profile that meets your model preference.
- 5) Complete the True Definition online Rx.
- 6) Upload the scan data to the Midmark Communication Center.

\* Dentists may request a modeless monolithic restoration by selecting the NDX Florida (No Model) profile that has been previously added to the scanner menu (see step 2 above).



4520 Parkbreeze Ct. • Orlando, FL 32808 • 800.262.2547 • [NDXFlorida.com](http://NDXFlorida.com)



NDX<sup>®</sup>  Florida

# Midmark Mobile True Definition<sup>™</sup> Scanner

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Florida, please contact us directly at **800.262.2547** or **[di-floridalabs@nationaldentex.com](mailto:di-floridalabs@nationaldentex.com)**