

Midmark Mobile True Definition™ Scanner

Doctors with a Midmark Mobile True Definition™ scanner can send digital impressions directly to NDX Ideal through the Midmark Connection Center.

To send a file:

- 1) Log in to your Midmark Connection Center Account that was created when your scanner was installed.
If assistance is needed, please contact Midmark Digital Impression Customer Service at 937.526.8387.
- 2) Verify that NDX Ideal and NDX Ideal (No Model) are available under the Scanner Laboratory menu.
 - If not, please contact Midmark Digital Impression Customer Service (937.526.8387) and request that these profiles be added to the scanner menu.
- 3) Select the NDX Ideal profile from the drop-down menu that matches your case preference:
 - NDX Ideal (choose this option when requesting models)
 - NDX Ideal (No Model*) (choose this option when requesting modeless restorations)
- 4) Complete the intraoral scan and select the NDX Ideal profile that meets your model preference.
- 5) Complete the True Definition online Rx.
- 6) Upload the scan data to the Midmark Communication Center.

* Dentists may request a modeless monolithic restoration by selecting the NDX Ideal (No Model) profile that has been previously added to the scanner menu (see step 2 above).



5501 Wilshire NE, Suite B • Albuquerque, NM 87113 • 800.998.6684 • NDXIdeal.com



NDX[®]  Ideal

Midmark Mobile True Definition[™] Scanner

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Ideal, please contact us directly at **800.998.6684** or **di-ideal@nationaldentex.com**