



Doctors with an iTero® scanner can send digital impressions directly to NDX Ito & Koby through iTero online.

## To send a file:

### iTero® Element™

- 1) Add **NDX Ito & Koby** (ID # **3565**) by calling iTero Support 800-577-8767.
- 2) Start a new Rx, and after entering patient name and case type, select *NDX Ito & Koby* from the Ship To menu.
- 3) If model-less, specify in the notes.
- 4) After scanning, press the *Envelope icon*, add signature, and press *Confirm & Send*.

### iTero® HD2.9 and Previous Models

- 1) Add **NDX Ito & Koby** (ID# **3565**) by calling iTero Support at 800-577-8767 or from within your iTero online account.
- 2) Start a new Rx, enter required information and select *NDX Ito & Koby* from the Ship To menu.
- 3) If model-less, specify in the notes.
- 4) After completing the scan process, press the *Send* button.



NDX<sup>®</sup>  Ito&Koby  
YOUR DENTAL LAB PARTNER

iTero<sup>®</sup>

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Ito & Koby, please contact us directly at **800.288.6684** or **di-ItoKoby@NationalDentex.com**.