


## To send a file:

- 1) Create an account on [www.sirona-connect.com](http://www.sirona-connect.com) and follow the prompts for activating the account (if you already have an account, proceed to step 3).
- 2) After account activation, log into your account and add **NDX Kiess Kraft** to the list of Favorite Laboratories.
- 3) Log into your Cerec software and complete the digital scan.
- 4) After completing scan acquisition and marking the margin, proceed to the Connect step and enter Login info.
- 5) Review your order and restoration information then click *Enter Order Data*.
- 6) Once the upload is complete, select *NDX Kiess Kraft* and desired return date.
- 7) Click *Add Additional Information* and enter gender, age, comments, and upload any additional files for the case. If model-less, please specify.
- 8) Review information and click *Submit Cart*.
- 9) Enter your user name and password once again and the case submission is complete.

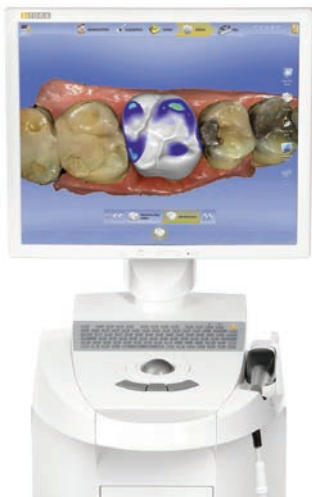

NDX<sup>®</sup>  Kiess Kraft  
YOUR DENTAL LAB PARTNER

Cerec<sup>®</sup>

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Kiess Kraft, please contact us directly at **800.553.9522** or **[di-KiessKraft@NationalDentex.com](mailto:di-KiessKraft@NationalDentex.com)**.