

# 3M™ True Definition Scanner

Doctors with a 3M True Definition scanner can send digital impressions directly to NDX Lumident through the 3M Connection Center.

## To send a file:

- 1) Log in to your 3M Connection Center Account that was created when your scanner was installed. If assistance is needed, please contact 3M Digital Impression Customer Service at 877.722.6528.
- 2) Verify that NDX Lumident and NDX Lumident (No Model) are available under the Scanner Laboratory menu.
  - If not, please contact 3M Digital Impression Customer Service (877.722.6528) and request that these profiles be added to the scanner menu.
- 3) Select the NDX Lumident profile from the drop-down menu that matches your case preference:
  - NDX Lumident (choose this option when requesting models)
  - NDX Lumident (No Model\*) (choose this option when requesting modeless restorations)
- 4) Complete the intraoral scan and select the NDX Lumident profile that meets your model preference.
- 5) Complete the True Definition online Rx.
- 6) Upload the scan data to the 3M Communication Center.

\* Dentists may request a modeless monolithic restoration by selecting the NDX Lumident (No Model) profile that has been previously added to the scanner menu (see step 2 above).



8840 Commerce Park Place, Bldg. 4 • Indianapolis, IN 46268 • 866.586.4336 • [NDXLumident.com](http://NDXLumident.com)



NDX<sup>®</sup>  Lumident

# 3M<sup>™</sup> True Definition Scanner

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Lumident, please contact us directly at **866.586.4336** or **[digital@lumident.net](mailto:digital@lumident.net)**