



Doctors using Cerec intraoral scanning systems can send digital impressions directly to NDX Lumident through Cerec Connect.

To send a file:

- 1) Create an account on www.sirona-connect.com and follow the prompts for activating the account (if you already have an account, proceed to step 3).
- 2) After account activation, log into your account and add **NDX Lumident** to the list of Favorite Laboratories.
- 3) Log into your Cerec software and complete the digital scan.
- 4) After completing scan acquisition and marking the margin, proceed to the Connect step and enter Login info.
- 5) Review your order and restoration information, and then click *Enter Order Data*.
- 6) Once the upload is complete, select *NDX Lumident* and the desired return date.
- 7) Click *Add Additional Information* and enter gender, age, comments, and upload any additional files for the case. If modeless, please specify.
- 8) Review information, and click *Submit Cart*.
- 9) Enter your user name and password once again, and then case submission is complete.



8840 Commerce Park Place, Bldg. 4 • Indianapolis, IN 46268 • 866.586.4336 • NDXLumident.com



NDX[®]  Lumident

Cerec[®]

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Lumident, please contact us directly at **866.586.4336** or **digital@lumident.net**

