

To send a file:

iTero Element:

- 1) Add **NDX Lumident** (ID# **16186**) by calling iTero Support at 800.577.8767.
- 2) Start a new Rx, and after entering patient name and case type, select *NDX Lumident* from the *Ship To* menu.
- 3) If modeless, specify in the notes.
- 4) After scanning, press the Envelope icon, add signature, and press *Confirm & Send*.

iTero HD2.9 and Previous Models

- 1) Add **NDX Lumident** (ID# **16186**) by calling iTero Support at 800.577.8767 or from within your iTero online account.
- 2) Start a new Rx, enter required information and select *NDX Lumident* from the *Ship To* menu.
- 3) If modeless, specify in the notes.
- 4) After completing the scan process, press the *Send* button.



NDX[®]  Lumident

iTero[®]

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Lumident, please contact us directly at **866.586.4336** or **digital@lumident.net**