

# Midmark Mobile True Definition™ Scanner

Doctors with a Midmark Mobile True Definition™ scanner can send digital impressions directly to NDX Rogers through the Midmark Connection Center.

## To send a file:

- 1) Log in to your Midmark Connection Center Account that was created when your scanner was installed.  
If assistance is needed, please contact Midmark Digital Impression Customer Service at 937.526.8387.
- 2) Verify that NDX Rogers and NDX Rogers (No Model) are available under the Scanner Laboratory menu.
  - If not, please contact Midmark Digital Impression Customer Service (937.526.8387) and request that these profiles be added to the scanner menu.
- 3) Select the NDX Rogers profile from the drop-down menu that matches your case preference:
  - NDX Rogers (choose this option when requesting models)
  - NDX Rogers (No Model\*) (choose this option when requesting modeless restorations)
- 4) Complete the intraoral scan and select the NDX Rogers profile that meets your model preference.
- 5) Complete the True Definition online Rx.
- 6) Upload the scan data to the Midmark Communication Center.

\* Dentists may request a modeless monolithic restoration by selecting the NDX Rogers (No Model) profile that has been previously added to the scanner menu (see step 2 above).



2407 Denso Drive • Athens, TN 37303 • 800.278.6046 • [NDXRogers.com](http://NDXRogers.com)



NDX<sup>®</sup>  Rogers

# Midmark Mobile True Definition<sup>™</sup> Scanner

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Rogers, please contact us directly at **800.278.6046** or **[di-rogers@nationaldentex.com](mailto:di-rogers@nationaldentex.com)**