

Midmark Mobile True Definition™ Scanner

Doctors with a Midmark Mobile True Definition™ scanner can send digital impressions directly to NDX Thoele through the Midmark Connection Center.

To send a file:

- 1) Log in to your Midmark Connection Center Account that was created when your scanner was installed.
If assistance is needed, please contact Midmark Digital Impression Customer Service at 937.526.8387.
- 2) Verify that NDX Thoele and NDX Thoele (No Model) are available under the Scanner Laboratory menu.
 - If not, please contact Midmark Digital Impression Customer Service (937.526.8387) and request that these profiles be added to the scanner menu.
- 3) Select the NDX Thoele profile from the drop-down menu that matches your case preference:
 - NDX Thoele (choose this option when requesting models)
 - NDX Thoele (No Model*) (choose this option when requesting modeless restorations)
- 4) Complete the intraoral scan and select the NDX Thoele profile that meets your model preference.
- 5) Complete the True Definition online Rx.
- 6) Upload the scan data to the Midmark Communication Center.

* Dentists may request a modeless monolithic restoration by selecting the NDX Thoele (No Model) profile that has been previously added to the scanner menu (see step 2 above).



2801 West St. Germain Street • St. Cloud, MN 56301 • 800.899.1115 • NDXThoele.com



NDX[®]  Thoele

Midmark Mobile True Definition[™] Scanner

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Thoele, please contact us directly at **800.899.1115** or **di-thoele@nationaldentex.com**