

# 3M™ True Definition Scanner

Doctors with a 3M™ True Definition scanner can send digital impressions directly to NDX Twin Cities through 3M Connection Center.

## To send a file:

- 1) Log in to your 3M Connection Center Account that was created when your scanner was installed. If assistance is needed, contact 3M Digital Impression Customer Service at 877.722.6528.
- 2) Verify that NDX Twin Cities and NDX Twin Cities (No Model), are available under the Scanner Laboratory menu.
  - If not, contact 3M Digital Impression Customer Service (877.722.6528) and request adding the profiles to the scanner menu.
- 3) Select the NDX Twin Cities profile from the drop down menu that matches your case preference:
  - NDX Twin Cities (choose this option when requesting models)
  - NDX Twin Cities (No Model)\* (choose this option when requesting model-less restorations)
- 4) Complete the intraoral scan and select the NDX Twin Cities profile that meets your model preference.
- 5) Complete the True Definition online Rx.
- 6) Upload the scan data to the 3M Communication Center.

\* Dentist may request a model-less monolithic restoration only by selecting the NDX Twin Cities (No-Model) profile that has been previously added to the scanner menu (see step 2 above).



NDX<sup>®</sup>  Twin Cities  
YOUR DENTAL LAB PARTNER

# 3M<sup>™</sup> True Definition Scanner

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Twin Cities, please contact us directly at **800.328.2568** or **[di-TwinCities@NationalDentex.com](mailto:di-TwinCities@NationalDentex.com)**.