



Doctors using Cerec intraoral scanning systems can send digital impressions directly to NDX nSequence through Cerec Connect.

To send a file:

- 1) Create an account on www.sirona-connect.com and follow the prompts for activating the account (if you already have an account, proceed to step 3).
- 2) After account activation, log into your account and add **NDX nSequence** to the list of Favorite Laboratories.
- 3) Log into your Cerec software and complete the digital scan.
- 4) After completing scan acquisition and marking the margin, proceed to the Connect step and enter Login info.
- 5) Review your order and restoration information, and then click *Enter Order Data*.
- 6) Once the upload is complete, select *NDX nSequence* and the desired return date.
- 7) Click *Add Additional Information* and enter gender, age, comments, and upload any additional files for the case. If modeless, please specify.
- 8) Review information, and click *Submit Cart*.
- 9) Enter your user name and password once again, and then case submission is complete.



6980 Sierra Center Parkway, Suite 100 • Reno, NV 89511 • 888.809.2777 • nsequence.com



NDX[®]  nSequence

Cerec[®]

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX nSequence, please contact us directly at **888.809.2777** or **wetransfer@sequencedental.com**