



Doctors with an iTero scanner can send digital impressions directly to NDX nSequence through iTero online.

To send a file:

iTero Element:

- 1) Add **NDX nSequence** (ID# **1574**) by calling iTero Support at 800.577.8767.
- 2) Start a new Rx, and after entering patient name and case type, select *NDX nSequence* from the *Ship To* menu.
- 3) If modeless, specify in the notes.
- 4) After scanning, press the Envelope icon, add signature, and press *Confirm & Send*.

iTero HD2.9 and Previous Models

- 1) Add **NDX nSequence** (ID# **1574**) by calling iTero Support at 800.577.8767 or from within your iTero online account.
- 2) Start a new Rx, enter required information and select *NDX nSequence* from the *Ship To* menu.
- 3) If modeless, specify in the notes.
- 4) After completing the scan process, press the *Send* button.



6980 Sierra Center Parkway, Suite 100 • Reno, NV 89511 • 888.809.2777 • nsequence.com



iTero®

NDX®  nSequence

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX nSequence, please contact us directly at **888.809.2777** or **wetransfer@sequencedental.com**