



Doctors with an iTero scanner can send digital impressions directly to NDX through iTero online.

To send a file:

iTero Element:

- 1) Add **your preferred NDX lab** by calling iTero Support at 800.577.8767.
- 2) Start a new Rx, and after entering patient name and case type, select **your preferred NDX lab** from the Ship To menu.
- 3) If model-less, specify in the notes.
- 4) After scanning, press the *Envelope icon*, add signature and press *Confirm & Send*.

iTero HD2.9 and Previous Models

- 1) Add **your preferred NDX lab** by calling iTero Support at 800.577.8767 or from within your iTero online account.
- 2) Start a new Rx, enter required information, and select **your preferred NDX lab** from the *Ship To* menu.
- 3) If model-less, specify in the notes.
- 4) After completing the scan process, press the *Send* button.

If you have any questions, please contact us at 561.721.9866.