



Doctors using Cerec® intraoral scanning systems can send digital impressions directly to NDX through Cerec Connect.

To send a file:

- 1) Create an account on www.sirona-connect.com and follow the prompts for activating the account (if you already have an account, proceed to step 3).
- 2) After account activation, log into your account and add **your preferred NDX lab** to the list of Favorite Laboratories.
- 3) Log into your Cerec software and complete the digital scan.
- 4) After completing scan acquisition and marking the margin, proceed to the Connect step and enter Login info.
- 5) Review your order and restoration information, and then click *Enter Order Data*.
- 6) Once the upload is complete, select **your preferred NDX lab** and the desired return date.
- 7) Click *Add Additional Information* and enter gender, age, comments, and upload any additional files for the case. If model-less, please specify.
- 8) Review information, and click *Submit Cart*.
- 9) Enter your user name and password once again, and then case submission is complete.

If you have any questions, please contact us at 561.721.9866.



1701 Military Trail Suite 155B • Jupiter, FL 33458 • NationalDentex.com